



Clinical guidelines

Preparation techniques

With Procera AllCeram you follow your normal procedures. You prepare and cement as you would with a regular core of metal alloy.

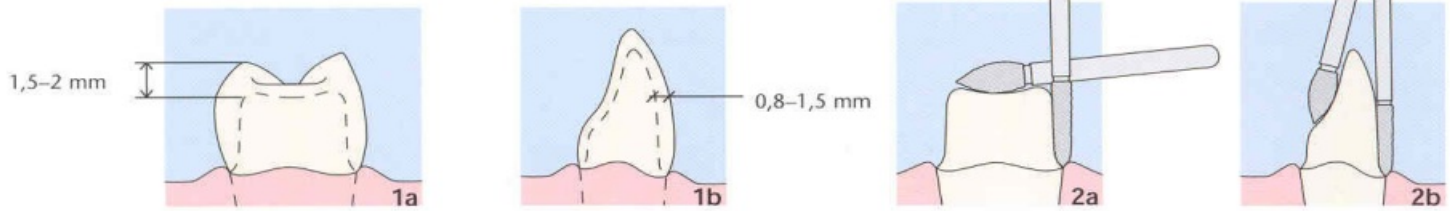

Recommended dimensions for preparation (figs. 1a and 1b). The preparation is characterized by thinned margins and a cutting with rounded inner angles and edges (figs. 2a and 2b).

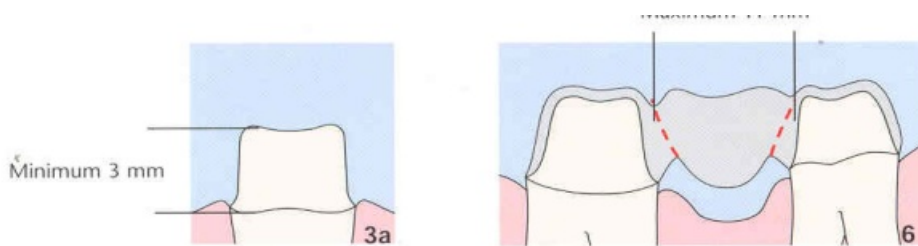

Bridge

General guidelines are identical to those for crowns with the following additions. The prepared teeth must allow a bridge with an occlusal-cervical height of at least 3 mm (fig. 5). The edentulous space between the unprepared teeth has to be less than 11 mm (fig. 6). The preparation should also be tapered away from the edentulous space to enable the correct connector design.



Laminate

The dimensions given in fig. 7a allow enough space for the porcelain to be created. The preparation can go down to a maximum of 2–3 mm on the palatal side (fig. 7b). The preparation can be extended to, or just pass, the contact points to a maximum of 1 mm (fig. 7c).